# SR Triplex<sup>®</sup> Hot

- [en] **Instructions for Use**  
Heat-curing denture base material
- [de] **Gebrauchsinformation**  
Heisspolymerisierendes Prothesenbasismaterial
- [fr] **Mode d'emploi**  
Résine dentaire de base thermopolymérisable
- [it] **Istruzioni d'uso**  
Materiale base per protesi dentali termopolimerizzabile
- [es] **Instrucciones de uso**  
Material base termopolimerizable para dentaduras
- [pt] **Instruções de Uso**  
Material para base de próteses, termopolimerizável
- [sv] **Bruksanvisning**  
Värmehärdande protesbasmaterial
- [da] **Brugsanvisning**  
Varmehærdende basismateriale til tandpleje
- [fi] **Käyttöohjeet**  
Lämpökovetteinen hammasproteesin runkomateriaali
- [no] **Bruksanvisning**  
Varmherdende protesebasismateriale
- [nl] **Gebruiksaanwijzing**  
Onder warmte uithardend basismateriaal voor gebitsprothesen
- [el] **Οδηγίες Χρήσης**  
Θερμοπολυμεριζόμενο υλικό βάσης οδοντοστοιχίας
- [tr] **Kullanım Talimatları**  
Isıyla sertleşen protez kaide materyali
- [ru] **Инструкция по применению**  
Материал горячей полимеризации для изготовления базисов зубных протез
- [pl] **Instrukcja stosowania**  
Termoutwardzalny materiał na płyty protez
- [sl] **Navodila za uporabo**  
Material osnove zobne proteze, ki se toplotno strjuje
- [hr] **Upute za uporabu**  
Toplo polimerizirajući materijal za bazu proteze
- [cs] **Návod k použití**  
Teplem tuhnoucí materiál na báze snímatelných zubních náhrad
- [sk] **Návod na používanie**  
Teplne vytvrdzovací základný materiál na zubné protézy
- [hu] **Használati utasítás**  
Hőre keményedő protézis-bázislemezanyag
- [sr] **Упутство за употребу**  
Топло полимеризујући материјал за основу протетске надокнаде
- [mk] **Упатство за употреба**  
Топлотно зацврстувачки материјал за база на протези
- [bg] **Инструкции за употреба**  
Топлинно полимеризиращ материал за протезна основа
- [sq] **Udhëzimet e përdorimit**  
Material baze proteze, me polimerizim me nxehtësi
- [ro] **Instrucțiuni de utilizare**  
Material pentru baza protezei cu polimerizare la cald
- [uk] **Інструкція із застосування**  
Матеріал із термічною полімеризацією для основи зубних протезів
- [et] **Kasutusjuhend**  
Kuumuse toimele kõvastuvate proteeside alusmaterjal
- [lv] **Lietošanas instrukcija**  
Karstumā cietējošs zobu protēžu bāzes materiāls
- [lt] **Naudojimo instrukcija**  
Termiškai kietėjanti protezų pagrindo medžiaga

CE 0123

Rx ONLY

Date information prepared:  
2024-07-26 / Rev. 0

**Manufacturer:**  
Ivoclar Vivadent AG  
Bendererstrasse 2  
9494 Schaan/Liechtenstein  
[www.ivoclar.com](http://www.ivoclar.com)

**ivoclar**

## 1 Intended use

### Intended purpose

- Fabrication of bases for removable dentures
- Fabrication of hard occlusal splints

### Patient target group

- Patients with permanent teeth
- Adult patients with dental implants
- Edentulous adult patients

### Intended users

- Dental laboratory technicians (fabrication of restorations in the dental laboratory)
- Denturists (fabrication of restorations in the dental laboratory)

### Special training

None

### Use

For dental use only.

### Description

Heat-curing denture base material for the packing technique.

### Technical specifications

Property	Value
Flexural strength	≥ 65 MPa
Flexural modulus	≥ 2000 MPa
Residual MMA content	≤ 2.2 %
Water absorption (7 days)	≤ 32 µg/mm <sup>3</sup>
Solubility (7 days)	≤ 1.6 µg/mm <sup>3</sup>

### Indications

- Partial edentulism in the anterior and posterior region
- Complete edentulism
- Dental dysfunctions and parafunctions

#### Types of restorations:

- Complete dentures
- Partial dentures
- Implant-supported dentures
- Occlusal splints

### Contraindications

The use of this product is contraindicated if the patient is known to be allergic to any of its ingredients.

### Limitations of use

- Direct contact of uncured material with intraoral tissues
- Use of the non-sterilizable shade guide on the patient
- Exceeding the maximum processing temperature of 110 °C / 230 °F
- Contact of polymerized PMMA with solvents
- Reuse of the restoration by other patients

### Side effects

Allergic reactions to methacrylate in individual cases

### Interactions

None known to date

### Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

### Composition

Polymer: Polymethyl methacrylate, dibenzoyl peroxide

Monomer: Methyl methacrylate, ethylene glycol dimethacrylate, unsaturated fats (peanut oil)

## 2 Usage

### 2.1 Fabricating the denture

#### Preparing the model

1. Soak the model in water for 5-10 min.
2. Isolate the model.
3. Allow the model to dry for 5 min.
4. Repeat the steps 2 to 3.

#### Investing the model and the waxed-up denture

1. Fill one of the two flask halves with stone.
2. Position the model 1 cm away from the flask margin.
3. Press the model down to the height of the flask margin.
4. Remove excess stone.
5. Allow the stone to set.

#### Casting the counter model

1. Cover the model and denture teeth with a heat-resistant silicone (Shore A hardness of >80).
2. Allow the silicone to dry.
3. Thinly isolate the stone-to-stone contact surfaces.
4. Assemble the flask.
5. Fill stone into the flask opening.
6. Allow the stone to set.

#### Boiling out and cleaning the flask

##### **NOTICE! Tooth necks and the basal tooth surfaces must be completely free of wax.**

1. Immerse the flask into the water bath (90 °C / 194 °F, 5-8 min).
2. Open the flask.
3. Remove coarse wax.
4. Boil out the wax.
5. Clean the flasks using clean, boiling water.
6. Roughen the basal tooth surfaces.

#### Isolating the stone surfaces

1. Allow the flask halves to cool down to <40 °C (<104 °F).
2. Isolate the surfaces of the plaster model, ensuring there is no pooling.
3. Wait for 5 min.
4. Repeat the steps 2 and 3.

#### Mixing and pressing the material

##### **NOTICE! Do not mix the material with other materials.**

##### **NOTICE! Observe the following processing parameters:**

Mixing ratio	Dough time	Working time at 23 °C / 73 °F	Polymerisation in water bath
23.4 g polymer : 10 ml monomer	10 min	Max. 20 min	<p><b>Average-sized denture (residual monomer content &lt;2.2 %):</b></p> <ol style="list-style-type: none"> <li>1. Place the closed flask in cold water.</li> <li>2. Heat the water up to 100 °C (212 °F).</li> <li>3. Polymerize the flask in the water bath for 45 min.</li> </ol> <p><b>Small/medium-sized dentures</b></p> <ol style="list-style-type: none"> <li>1. Place the closed flask in hot water.</li> <li>2. Heat the water up to 100 °C (212 °F).</li> <li>3. Polymerize the flask in the water bath for 20 min.</li> </ol>

Table 1

1. Thoroughly mix the monomer and polymer for 20-30 s to a homogeneous consistency in a mixing cup using a spatula.
2. Allow the material to swell according to Table 1.
3. Press the prepared material into one of the two flask halves.
4. Assemble the flask.
5. Secure the flask with screws, or the flask mounted in the clamping frame without screws, in the dental press (80 bar).
6. Close the screws of the flask or the clamping frame.
7. Polymerize the denture according to Table 1.

#### Cooling the flask

##### **NOTICE! Cool the flask as described. Cooling the denture too rapidly may cause tension cracks.**

1. Remove the flask using thermal protection gloves.
2. Allow the flask to cool at room temperature for 30 min.
3. Allow the flask to cool down in a water bath <15 °C (<59 °F) for 15-25 min.

#### Divesting the denture

##### **NOTICE! Risk of injury due to sharp press flash. Wear gloves.**

1. Open the flask screws or the clamping frame.
2. Remove the stone.
3. Divest the denture.

## Finishing the denture

**NOTICE! The processing of the material can result in sharp edges so that there is a risk of injury.**

1. Check the occlusion in the articulator.
2. Carefully remove the denture from the model.
3. Finish the denture.
4. Polish the denture to a high gloss.

## 2.2 Extending/relining/repairing the denture

1. Use cold-curing polymers.  Observe the instructions of the manufacturer.

## 3 Safety information

- The product has been developed solely for use in dentistry. Process according to the Instructions for Use.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, [www.ivoclar.com](http://www.ivoclar.com) and your responsible competent authority.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

## Warnings

### CAUTION

#### **Product contains methacrylates.**

Irritation of eyes, respiratory tract and skin  
Sensitization of skin

- a) Avoid skin contact with monomer and unpolymerized material (commercially available medical gloves do not provide adequate protection).
- b) Use an extraction unit and wear protective goggles and a face mask.

### CAUTION

#### **Product contains methyl methacrylate (MMA). MMA is highly flammable.**

Fire hazard (flash point + 10 °C / 50 °F)

- a) Keep away from sources of ignition.
- b) Take precautionary measures against electrostatic discharge.

## Supporting documents

Document	To be found at:
Current version of Instructions for Use	<a href="http://www.ivoclar.com/eIFU">www.ivoclar.com/eIFU</a>
Structure of Instructions for Use and Warnings	<a href="http://www.ivoclar.com/eIFU">www.ivoclar.com/eIFU</a>
Explanation of symbols	<a href="http://www.ivoclar.com/eIFU">www.ivoclar.com/eIFU</a>
Safety Data Sheet (SDS)	<a href="http://www.ivoclar.com">www.ivoclar.com</a>

## Disposal information

Dispose contaminated products, remaining stocks or removed restorations according to the corresponding national legal requirements. The product or product residues must not reach the sewage system.

## Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following known residual clinical risks exist:

- Ingestion of fragments
- Occurrence of denture stomatitis

## 4 Shelf life and storage

- Storage temperature 2–28 °C (36–82 °F)
- Do not use the product after the indicated expiry date.
- Protect the product from direct sunlight.

## 5 Additional information

Keep material out of the reach of children!

The manufacturer accepts no liability for damages resulting from a failure to observe the Instructions for Use. Furthermore, the user is responsible for testing the products for their suitability for any purpose not explicitly stated in the Instructions.

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